



# OASIS™


## LSO FITTING INSTRUCTIONS

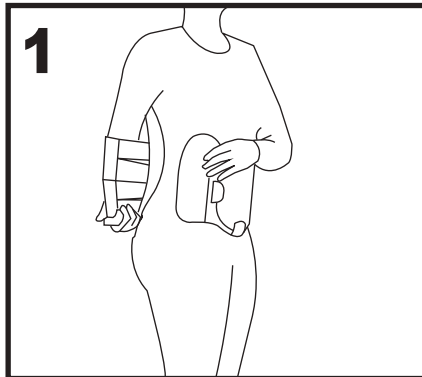


**1** Place the orthosis around your waist with the side closure pre-attached to the left side of the front panel. Proceed to attach the right hook and loop attachment to the right side of the front panel.

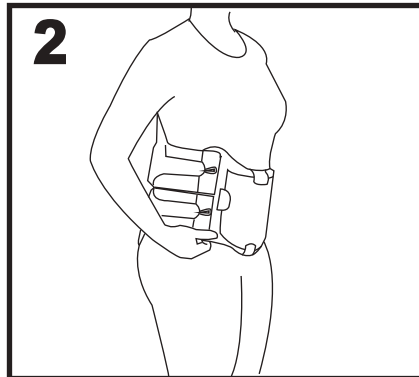


**2** Align the orthosis so that front panel is centered over your abdomen and the center of the back panel is aligned over your spine.

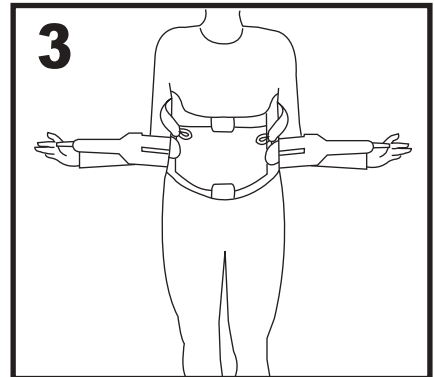


**3** Using the arthritic thumb grips, detach the bottom compression straps and pull straps simultaneously away from your body.

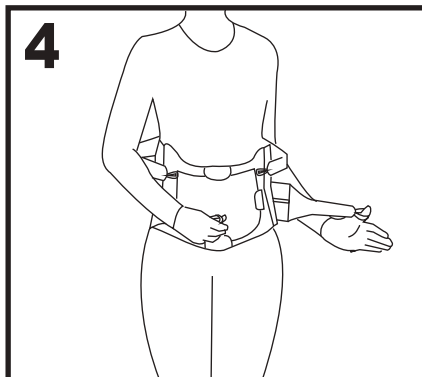

**4** Secure the left strap to the center of the orthosis followed by the right strap. Repeat for top strap if applicable.

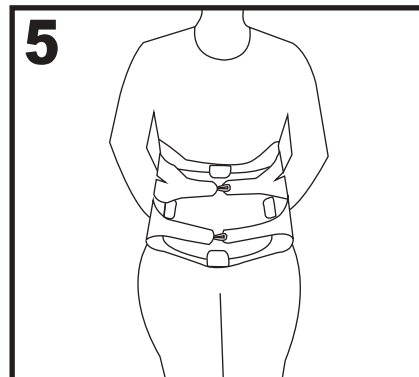

**5** Finally, ensure that the orthosis fit is snug and does not cause any discomfort.

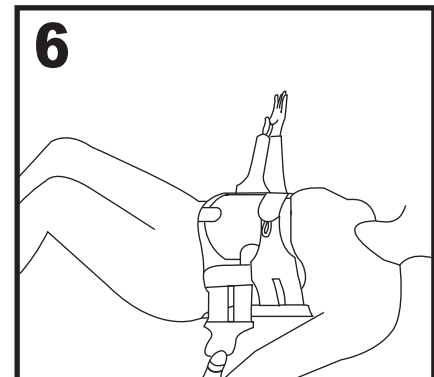

**6** For best results, re-tighten the straps while lying on a bed with your knees bent and feet flat.

### Care of your Oasis™

Always wear a t-shirt or orthopedic body sock under your orthosis. To clean the orthosis, hand wash it with mild soap in cool water. Allow the orthosis to air dry. It is not uncommon for your body to change size while wearing the orthosis because of the compression and reduction in post-surgical swelling. If you change enough in size that you can no longer adjust the straps to achieve compression, see your practitioner for a follow-up visit to resize or replace your brace.

©Copyright ©2006 Velcro Industries B.V. All rights reserved. Velcro Industries B.V., VELCRO® and VELCRO with the FLYING-1® (DESIGN) are registered trademarks of Velcro Industries B.V.

#### SIZE:

XS (31" - 37")  S (35" - 41")  M (39" - 45")  L (43" - 49")  XL (47" - 53")  2X (51" - 57")  3X (55" - 61")

#### MODEL:

LSO  TLSO ATE  TLSO DLK  
 LSO LP  TLSO LP ATE  TLSO LP DLK  
 TLSO  TLSO 4 PANEL  
 TLSO LP 4 PANEL

ATTACHMENTS:  DLK  PPK  SPK  PPX  SPX



Ph: 888-982-8181 • Fx: 877-288-4197 • [www.optecusa.com](http://www.optecusa.com)