





SINGLE OPENING


FITTING INSTRUCTIONS

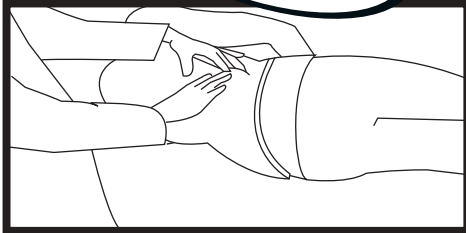


1. FOR BOTH PRACTICAL AND MEDICAL REASONS, PATIENTS REQUIRE ASSISTANCE IN DONNING THE ORTHOSIS. THE ASSISTANT MUST CAREFULLY FOLLOW EACH STEP. Begin with the patient lying down on his side. While holding the brace open, carefully log-roll the patient into the orthosis.

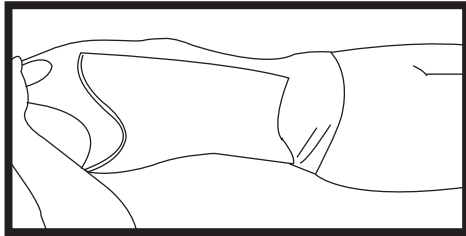

2. IMPORTANT: Adjust the position of waistline grooves of the shell so that it is oriented just above the pelvis but below the ribs, while ensuring that the undershirt is smooth underneath the brace. This prevents the skin from being pinched. While leaning against a wall, wrap the brace around the torso, with the opening centered on the abdomen.

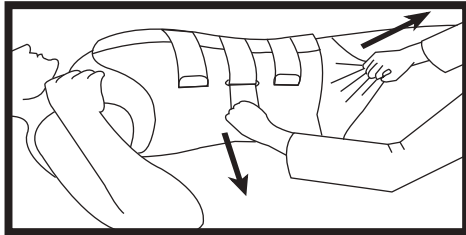

3. Tighten the Velcro straps, beginning with the center strap and finishing with the top strap. While tightening the top strap, pull up in a diagonal direction on the undershirt. Again, this prevents the patient's skin from being pinched.

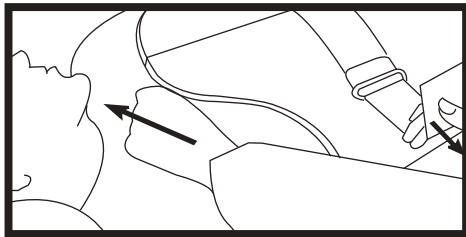

4. For best results, re-tighten straps with the patient laying on a flat surface with knees bent and feet flat. Finally ensure that orthosis fit is snug and does not cause any discomfort.

Care & General Guidelines

Patients are advised to purchase snug-fitting (a size smaller than the usual) undershirts to wear under the orthosis. For the duration of acute care, nursing must periodically remove the orthosis to check the patient's skin. Hand wash at 86 F with mild soap. When drying, do not expose to direct heat (heater or sunlight); air dry. Never iron or dry clean. If not rinsed well after washing, residual soap may cause skin irritation and product deterioration. It is not uncommon for your body to change size under the spinal brace, because of the compression of the brace and because of a reduction in post-surgical swelling. If you change enough in size that you can no longer adjust the straps enough to get good compression, see your practitioner for a follow-up visit to resize or replace your brace. Your practitioner knows your medical status and is using the spinal brace as one part of your total care. It is important to follow his or her instructions exactly, even if they differ from some of the preceding guidelines.

NOTE: Special care must be taken with insensate patients to avoid causing pinching along the mid-axillary line, especially in the waistline region.

WARRANTY: The warranty on the Single Opening LSO or TLSO is free repair or replacement for manufacturer's defects within 90 days of the date fitted.

Ph: 888-982-8181 Fx: 877-288-4197 • www.optecusa.com