



TLSO w/ ATE PATIENT FITTING INSTRUCTIONS



1
Begin with the Stealth Corset open and the strap on the Thoracic Attachment closed on the left side only. Your caregiver should have already adjusted the Thoracic Attachment and the position of the front panel specifically to your body.

2
Slip left your left arm through the loop of the Thoracic Attachment so that the strap rests on your shoulder and the Rigid Front Panel is against your abdomen.

3
Center the Back Plastic Panel on your back and position the waist grooves at your waist. Loosely attach the Stealth Corset by attaching the Right Panel to the Left Panel. Snap the right strap of the Thoracic Attachment into the buckle.



4
Re-open the Stealth Corset by detaching the Right Panel from the Left Panel. Align the Left Panel of the Corset with the Rigid Front Panel attached to it on your abdomen and press firmly with your left hand. Pull Right Panel of Corset with velcro snugly over the left velcro attachment.

5
Using the arthritic thumb grips, detach the bottom compression straps for orthosis that have two straps on each side, or the single strap for orthosis that have only one strap on each side, and pull straps simultaneously away from your body. Secure the left strap to the center of the orthosis followed by the right strap. Repeat for top straps, if applicable.

6
For best results, re-tighten straps in the same order while laying on a flat surface with your knees bent and feet flat. Finally, ensure that orthosis fit is snug and does not cause any discomfort.

Care & General Guidelines

Before cleaning, remove plastic inserts from front and back panels. Hand wash at 86 F with mild soap. When drying, do not expose to direct heat (heater or sunlight) air dry. Never iron or dry clean. If not rinsed well after washing, residual soap may cause skin irritation and product deterioration. It is not uncommon for your body to change size under the spinal brace, because of the compression of the brace and because of a reduction in post-surgical swelling. If you change enough in size that you can no longer adjust the straps enough to get good compression, see your practitioner for a follow-up visit to resize or replace your brace. Your practitioner knows your medical status and is using the spinal brace as one part of your total care. It is important to follow his or her instructions exactly, even if they differ from some of the following guidelines.

WARRANTY: The warranty on the Stealth TLSO is free repair or replacement for manufacturer's defects within 90 days of the date fitted.

©Copyright ©2006 Velcro Industries B.V. All rights reserved. Velcro Industries B.V. VELCRO® and VELCRO with the FLYING-V® (DESIGN) are registered trademarks of Velcro Industries B.V.



"Orthotics on Time"

STEALTH DESIGN:	<input type="checkbox"/> TLSO ²	THORACIC ATTACHMENT:	<input type="checkbox"/> SPK	<input type="checkbox"/> PPK	OPTIONAL COMPONENTS:	<input type="checkbox"/> Hot/Cold Pack
	<input type="checkbox"/> TLSO ⁴		<input type="checkbox"/> SPX	<input type="checkbox"/> PPX		<input type="checkbox"/> Rehab Panel

SIZE	WAIST MEASUREMENT	QTY.	TLSO ⁴ - WHITE MODEL #	QTY.	TLSO ² - WHITE MODEL #	QTY.	TLSO ² - WHITE MODEL #	QTY.	TLSO ² - BLACK MODEL #
SMALL	23" - 30"		STT4 __ _WSM		STT2 __ _WSM		STT2 __ _WSM		STT2 __ _BSM
MEDIUM	27" - 34"		STT4 __ _WMD		STT2 __ _WMD		STT2 __ _WMD		STT2 __ _BMD
LARGE	32" - 40"		STT4 __ _WLG		STT2 __ _WLG		STT2 __ _WLG		STT2 __ _BLG
X-LARGE	37" - 44"		STT4 __ _WXL		STT2 __ _WXL		STT2 __ _WXL		STT2 __ _BXL
2X-LARGE	41" - 48"		STT4 __ _W2X		STT2 __ _W2X		STT2 __ _W2X		STT2 __ _B2X
3X-LARGE	45" - 52"		STT4 __ _W3X		STT2 __ _W3X		STT2 __ _W3X		STT2 __ _B3X
4X-LARGE	49" - 56"		STT4 __ _W4X		STT2 __ _W4X		STT2 __ _W4X		STT2 __ _B4X