



## MAINTENANCE AND CARE

- Brace should be worn over a cotton shirt.
- Fold the brace and attach the handle near the patient's lateral midline.
- Hand wash in warm water, towel off, and air dry in shade. Do not use any substances containing chlorine.

AT THE END OF ITS USE LIFE, DISCARD IT ALONG WITH YOUR NORMAL WASTE

## INDICATIONS

- Burst FX
- Post-Op Spinal Lumbar Instability
- Post-Op Disc Surgery
- Post-Op Laminectomy
- Kyphosis
- Scoliosis
- Paraplegic
- Cerebral Palsy
- Degenerative, Herniated or Bulging Disc
- Spondylolysis
- Spondylolisthesis
- Facet Syndrome

- Spinal Stenosis
- Tumors or Inflammation
- Chronic back pain
- Spinal Instability

And other conditions determined by an orthopedic surgeon, orthotist or other healthcare professional

Subscribe to our Fitting instructions videos on YouTube at:  
[www.youtube.com/OPTECUSA](http://www.youtube.com/OPTECUSA)

### OPTEC USA

975 Progress Circle Lawrenceville GA 30043  
 International  
 Ph: 1.770.513.7380 • F: 1.770.513.4365  
 Ph: 888.982.8181 • F: 877.288.4197  
 Email: [sales@optecusa.com](mailto:sales@optecusa.com)



[www.optecusa.com](http://www.optecusa.com)

# TLSO w/ DLK



## FITTING INSTRUCTIONS



**Warranty:** OPTEC USA products are warranted for free repair or replacement due to manufacturer's defects within 90 days of the date of purchase or the date fitted.