


The Venum® Hybrid is the ultimate one-size-fits-all soft comfort brace for the lumbar-sacral region. Its primary function is to reduce pain, provide support and immobilization to the lumbar-sacral spine. (See indications on the other side).

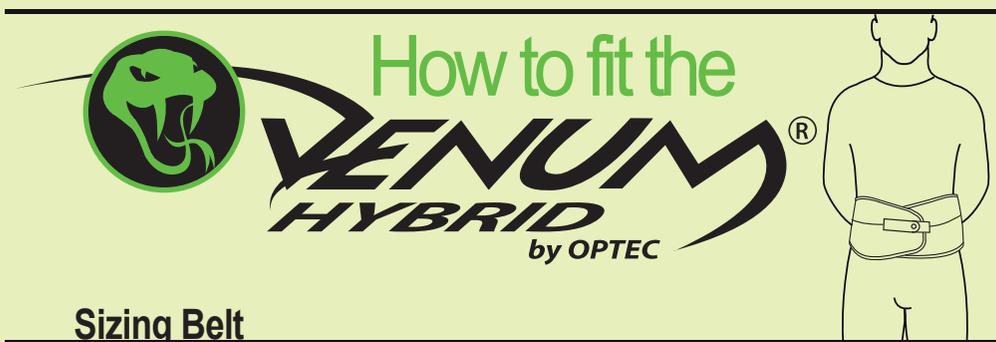

Sizing Belt

- 1 Detach pull handles off soft panel surface starting point.
- 2 Open clamp on each side and detach panels by pulling up and away from hook surface.
- 3 Set adjustment indicator window over desired size, fold neatly and press down firmly.
- 4 Place belt panels back in clamp and press down firmly.
- 5 Place handles back on soft panel surface starting point.
- 6 Pull ends in opposite directions to reset locking mechanism.

Choosing your Panels


- 7 VENUM HYBRID offers A DUAL OR SINGLE COMPRESSION CLOSURE DESIGN.
- 8 VENUM HYBRID offers two panels design configurations: CENTRAL OR INDEPENDENT SUPPORT PANELS
- 9 To fit the VENUM HYBRID LITE & PRO: Choose the desired panel (INDEPENDENT OR CENTRAL) place or slide panel to the center of the brace and press down firmly.
- 10 To fit the VENUM HYBRID X: Choose the desired panel (INDEPENDENT OR CENTRAL) place or slide panel to the center of the brace and press down firmly. Slide lateral panels thru the left and right side of the brace and press down firmly
- 11 The Venum Hybrid also offers an extender panel that adds 10" to the circumference of the waist.

Fitting Brace


- 12 Position brace around your back, with back plastic panel(s) centered on your spine and the curve on back panel directly over your spinal curvature.
- 13 Place left soft belt panel against your abdomen.
- 14 Place right soft belt panel with hook interface over left panel snug and press down firmly.
- 15 Remove reel from pull handle pocket, and decrease or increase length by reeling cable in or out, respectively.
- 16 Pull ergonomic handles, outward to achieve desired compression and fasten onto front soft panel (With desired compression, pull handles end up near midline, otherwise repeat step 15).

Warranty: OPTEC USA products are warranted for free repair or replacement due to manufacturer's defects within 90 days of the date of purchase or the date fitted.

Sizing Belt

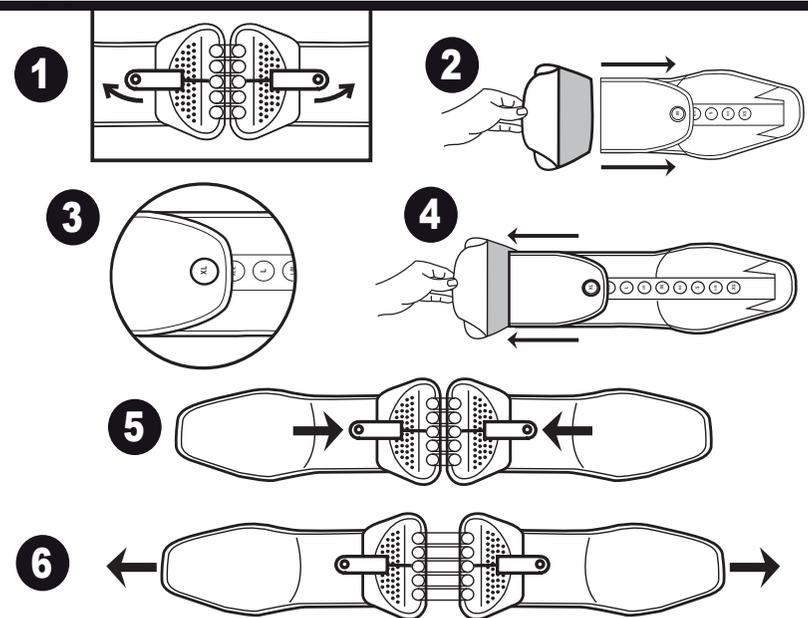

Choosing your Panels

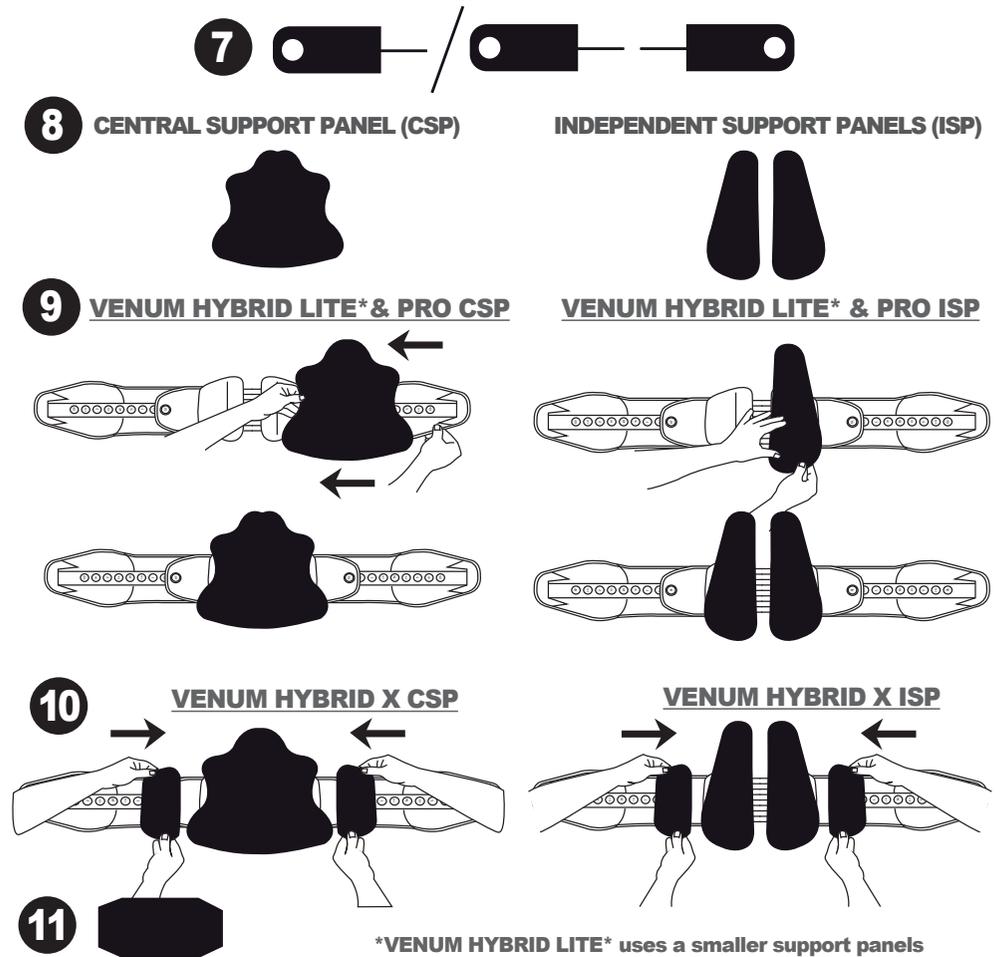

Fitting Brace

